



740SELECT™ Capnography: **Nomoline™ Adapter Sampling Line**

The Nomoline “No Moisture” Sampling Line is used exclusively with the Masimo ISA™ Sidestream CO₂ analyzer. As a system, the ISA CO₂ analyzer and Nomoline deliver the highest CO₂ monitoring performance in an ultra-compact design.

The Masimo ISA CO₂ analyzer has been fully integrated with the CASMED 740SELECT Series of patient monitors.

The 740SELECT supports standard of care requirements offering dual parameter (etCO₂ and SpO₂) and multi-parameter configurations for use in a wide variety of clinical settings.

Parameters available with the 740SELECT include: etCO₂, SpO₂, NIBP, Predictive or Temporal Artery Temperature, Masimo rainbow® Parameters (RRa™, SpCO®, PVI®) and 3-Lead ECG.

Nomoline Adapter Sampling Line Features

Reduces per Patient Cost:

- Allows use of non-proprietary cannulas
- Provides extended use in both low and high humidity environments

No Cross Contamination

The Nomoline contains a bacterial filter with a filtration efficiency of ≥ 99.9980% protecting the analyzer from water intrusion and cross contamination.


Eliminates the Need for a Water Trap:

Patented polymer design allows water in the sampling line to evaporate into the surrounding air; leaves oxygen and carbon dioxide gases unaffected.

Specially Designed for Low-Flow Applications:

- Excellent response time, making CO₂ measurement possible at high respiratory rates (up to 150 bpm)
- For use on all types of patients from adults to infants

Nomoline Design

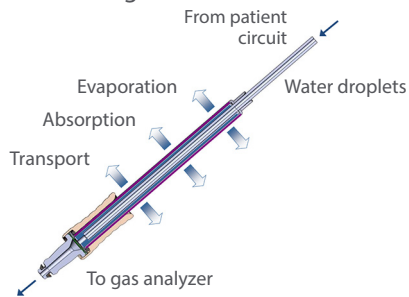

Nomoline fluid protection technology incorporates a unique sampling tube made of a special polymer water separation section - the *nomo* section and a bacterial filter. Unlike traditional solutions that remove water vapor or collect water in a plastic container, the Nomoline removes aspirated or condensed water. Water passes through the membrane-like surface of the Nomoline and evaporates into the surrounding air, while leaving oxygen, and carbon dioxide gases unaffected.

Masimo ISA CO₂ Sidestream Analyzer Monitoring Solutions

	01-03-0256 Qty. 1	ISA CO₂ Sidestream Analyzer (800101) • Plug-in and measure Sidestream CO ₂ analyzer • Adult/Pediatric/Infant <i>Notes: Optional software required to enable CO₂ monitoring on the 740SELECT monitor; Requires 740SELECT interface cable 01-02-0494</i>
	01-02-0488 Box of 25	ISA Nomoline Adapter (108220) • Sampling line with female luer lock connector • With integrated water removal and hydrophobic bacteria filter • Adult/Pediatric/Infant • Multi-patient use
	01-02-0844 Box of 25	ISA Nomoline Adapter with Male Luer Lock Connector (108210) • With integrated water removal and hydrophobic bacteria filter • For general use • Length 2 meters • Single patient use
	01-02-0845 Box of 20	ISA Nomoline Adapter Airway Adapter Set (108230) • Sampling line with straight airway adapter • Adult/Pediatric • Single patient use • Integrated water removal and hydrophobic bacteria filter • 2 meters
	01-02-0848 Box of 25	ISA T-Adapter (108250) • Airway adapter with female luer lock connector • Adult/Pediatric • Single patient use
	01-02-0865 Box of 25	Adult - Salter Labs® CO₂ Cannula Sample Line (4000) • 7' (2.1 meters) supply tube, male luer connector • Single patient use • Latex free
	01-02-0867 Box of 25	Pediatric - Salter Labs CO₂ Cannula Sample Line (4100) • 7' (2.1 meters) supply tube, male luer connector • Single patient use • Latex free
	01-02-0869 Box of 25	Infant - Salter Labs CO₂ Cannula Sample Line (4200) • 7' (2.1 meters) supply tube, male luer connector • Single patient use • Latex free
	01-02-0871 Box of 25	Intermediate Infant - Salter Labs CO₂ Cannula Sample Line (4300) • 7' (2.1 meters) supply tube, male luer connector • Single patient use • Latex free
	01-02-0494 Qty. 1	740SELECT Interface Cable (ISA/IRMA) • Used to interface an ISA or IRMA CO ₂ analyzer with a 740SELECT monitor

Reference 21-05-0255 for a complete list of ISA CO₂ accessories.

Contact Zoe Medical customer support for ordering information at customersupport@zoemedical.com.

Dealer Imprint

740SELECT is a trademark of Zoe Medical, Inc. Masimo®, rainbow®, SpCO®, PVI®, ISA™, Nomoline™, and RRA™ are registered trademarks of Masimo Inc. Zoe Medical reserves the right to make changes to this document without notice.